



# Sofia<sup>®</sup> SARS Antigen FIA

AVAILABLE FOR SALE IN THE USA AS EUA  
FOR BOTH SOFIA 2 AND SOFIA INSTRUMENTS

## Objective SARS-CoV-2 Results at Your Fingertips



Accurate, objective and automated results in **15 minutes**



Track via **secure, remote instrument management** with **Virena<sup>®</sup>**



**Excellent performance** compared to molecular methods



**Integrated data management** automatically stores test and user history



**Flexible, dual mode testing** for high throughput in variety of laboratory environments




User and patient ID captured with **onboard barcode scanner**




# Sofia SARS Antigen FIA – Procedure

## Nasal/Nasopharyngeal Swab Procedure

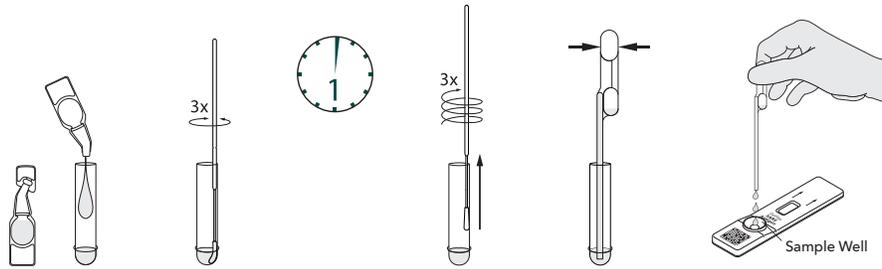


## Sofia Development Modes

### WALK AWAY Mode – Walk away and multitask

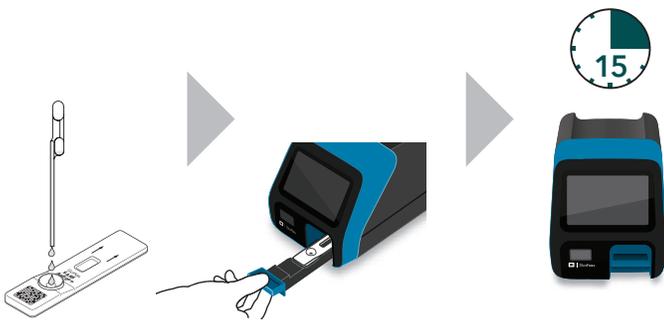

### READ NOW Mode – Batch multiple samples per hour

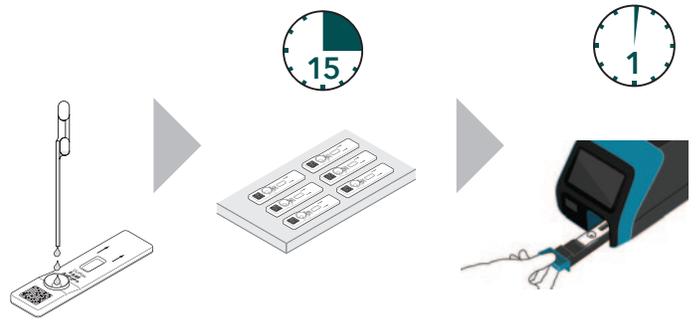

## Sofia SARS Antigen FIA

- Nasal and nasopharyngeal specimens
- Accurate detection with direct samples
- Results in 15 minutes
- Kit includes 25 sterile nasal swabs and a set of positive and negative control swabs
- Room temperature storage

## Sofia SARS Antigen FIA Clinical Performance\*

	Reference Extracted RT-PCR assay				95% CI			
	POS	NEG	Total	PPA	96.7%	83.3%	99.4%	
<b>Sofia SARS Antigen FIA Assay</b>	POS	29	0	29	NPA	100.0%	97.9%	100.0%
	NEG	1	179	180	PPV	100.0%	88.3%	100.0%
	Total	30	179	209	NPV	99.4%	96.9%	99.9%
					Prevalence	10.3%	7.4%	14.3%
					% agreement	99.7%		

\*Please refer to the Package Insert for full study design and explanation.

