

Nerivio®

GETTING STARTED

Using your Nerivio® device


1

Download and install the Nerivio app



2

Connect the device to the app

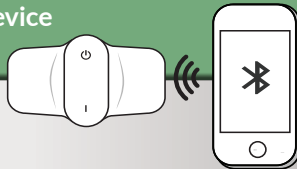

3


Place the device on your upper arm



4

Start Nerivio treatment within 60 minutes of the start of a migraine attack and complete the full 45-minute treatment.

Set the treatment intensity level so it feels strong yet comfortable and not painful.

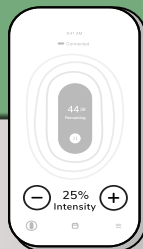

Nerivio®

Cares Team
is here to help!

Phone: 937- Nerivio® (637-4846)

Email: support@nerivio.com



INDICATION FOR USE: Nerivio is indicated for acute treatment of migraine with or without aura in patients 12 years of age or older. It is a prescription use, self-administered device for use in the home environment at the onset of migraine headache or aura.

IMPORTANT SAFETY INFORMATION: Nerivio should not be used by people with congestive heart failure, severe cardiac or cerebrovascular disease, or uncontrolled epilepsy. Nerivio should not be used by anyone with active implantable medical devices (e.g. pacemaker, hearing aid implant). It should only be applied on the upper arm over dry, healthy skin with normal physical sensation and without any metallic implants or in proximity to cancerous lesions. Nerivio has not been evaluated in pregnancy or those under the age of 12. For full use instructions and safety information, please see the Nerivio User Manual.


Now eligible for
insurance coverage
Nerivio^{Express}
SAVINGS PROGRAM

Co-pay for first Nerivio device is
\$10 for insured and eligible patients.



Nerivio®

A drug-free, prescribed wearable therapeutic device for the acute treatment of migraine

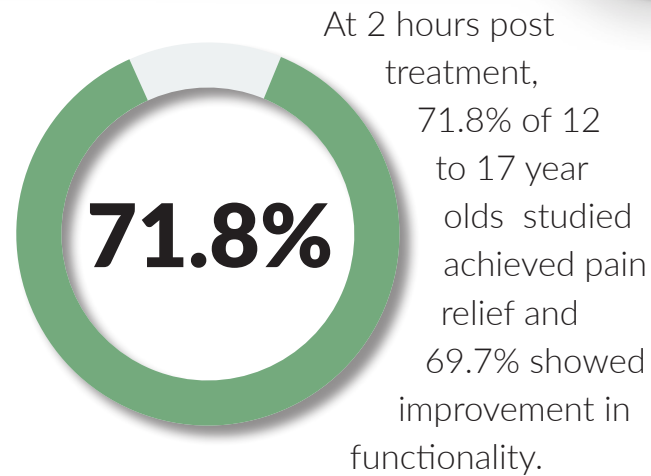

Nerivio *Express* SAVINGS PROGRAM



Get a **NERIVIO PRESCRIPTION** from your healthcare provider



Call Nerivio Express at **877-210-1206** if you do not hear from the pharmacy within 24 hours




Nerivio Express will ship your Nerivio device to your door!!



Nerivio®


INSTRUCTIONAL VIDEOS

Treating a migraine headache with NERIVIO

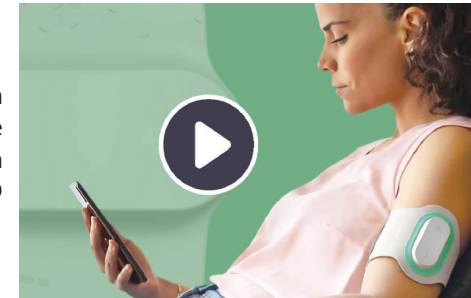

Important instructions for effective treatment

Using MyNerivio Diary & MyAnalytics

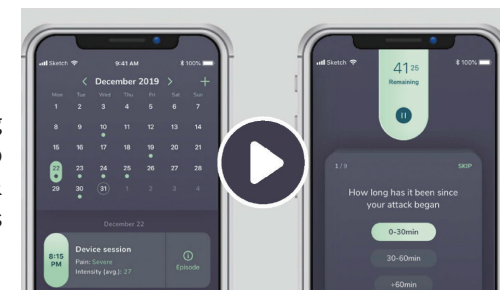

Scan QR Code for Instructional Videos

*Hershey AD, Lin T, Gruper Y, et al. Remote electrical neuromodulation for acute treatment of migraine in adolescents [published online ahead of print, 2020 Dec 21]. *Headache*. 2020;10.1111/head.14042. doi:10.1111/head.14042