


AUTOCON® III VET – *Now With Vessel Sealing*

Versatile electrosurgical unit for use
in small, large & exotic animals




AUTOCON® III VET

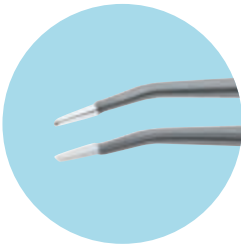

Open Surgical
Procedures



NEW
Vessel Sealing

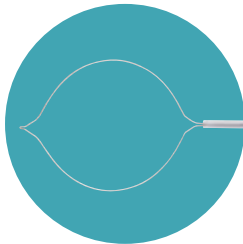


Polypectomy,
Gonadectomy, etc



Bipolar Cutting &
Coagulation



Arthroscopic
Vaporization

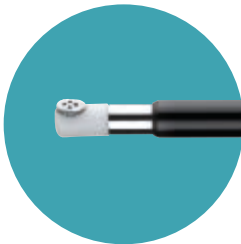

AUTOCON® III VET

The AUTOCON® III VET is a versatile electrosurgical unit that can be used in bipolar or unipolar mode for a variety of surgical procedures. The NEW BIVASCULARSAFE setting delivers permanent sealing of vessels up to 7 mm in diameter. The AUTOCON® III VET is the ideal solution for precise coagulation, cutting and fusion of tissues in endoscopic as well as open surgical procedures.

The AUTOCON® III VET is a multidisciplinary high frequency unit that can be used in a wide variety of veterinary applications, including:

- Laparoscopic transection of ovarian pedicle or other structures
- Endoscopic transluminal mass resection
- Hemostasis
- Arthroscopic contouring, cutting, hemostasis and capsular shrinkage
- Celioscopic gonadectomy in birds and reptiles
- Theloressectoscopy in dairy cattle
- Laryngeal surgery in horses
- Polypectomy
- Cutaneous mass removal

Features:

- NEW BIVASCULARSAFE Mode for coagulation, cutting and tissue fusion
- Audible signal indicates tissue is sealed
- Preprogrammed Unipolar and Bipolar settings
- Custom procedure settings can be created and stored
- High Definition 9" color touch screen
- Intuitive, user friendly control
- Precise power settings enable optimal energy delivery

Check out our schedule of upcoming hands-on training courses at
<http://go.karlstorz.com/eventsVET>



AUTOCON® III VET

Ordering Information



UH301E **AUTOCON® III VET**, BIVASCULAR, power supply 220 – 240 VAC, 50/60 Hz, including mains cord

HF connecting sockets unipolar:

1x 3-pin US type

1x 5 mm connector KARL STORZ/Erbe VIO

HF connecting sockets bipolar:

1x 2-pin US type (28.58)

2x KARL STORZ/Erbe VIO

Connecting socket:

Neutral electrode 2-pol

Can be integrated into OR1™ with KARL STORZ OR1™ SCB control WU300

System requirements: SCB R-UI Software Release 20090001-46 or higher

UH301U **Same**, power supply 100 – 127 VAC

Optional Footswitches



UF901 **One-Pedal HF Footswitch**, with button for switchover function, for use with HF generators



UF902 **Two-Pedal HF Footswitch**, with button for switchover function, for use with HF generators

Vessel Sealing Instrument for use with the AUTOCON® III VET BiVascularSafe Handpiece

Our new single use BiVascularSafe laparoscopic vessel sealing instrument delivers reliable sealing of vessels of up to 7 mm in diameter. The long 22 mm jaws ensure big tissue bites, while the instrument delivers uniform pressure, ensuring consistent results.

- Single use instruments improve patient safety – no cross contamination and performs as intended
- 220° Jaw rotation
- Pre-sterilized
- Audible feedback



60651VS

BIVASCULAR VET, laparoscopic ligation instrument, single action jaws, size 5 mm, length 34 cm, sterile, for single use, package of 5

ROBI® Rotating Bipolar Instruments for Laparoscopic Interventions

Our line of ROBI® bipolar instruments deliver reliable coagulation and cutting during laparoscopic procedures. Combining ROBI® instruments with the AUTOCON® III VET makes laparoscopic interventions easier, faster and safer for the patient.

All ROBI® instruments are modular and can be rotated 360°. They are easy to use, easy to clean and easy to assemble.

ROBI® PLUS 2, reusable bipolar forceps, delivers safe coagulation and cutting with a single 5 mm instrument.



66351DN **ROBI PLUS® 2**, rotating, with connector pin for bipolar coagulation, with integrated cutting function, single action jaws, size 5 mm, length 35 cm including:
Ring Handle
Outer Sheath
Knife Sheath
Forceps Insert

66451DN **Same**, length 42 cm



66300M **Knife Sheath with Integrated Blade**, for single use, length 35 cm, for use with Bipolar Instrument ROBI PLUS® 2 66351DN

66400M **Same**, length 42 cm

ROBI® Instruments, bipolar, 5 mm



38651MZ

ROBI® Scissors, CLERMONT-FERRAND model, rotating, dismantling, with connector pin for bipolar coagulation, single action jaws, straight jaws, serrated, for cutting and bipolar coagulation, size 5 mm, length 36 cm including:

Plastic Handle, without ratchet

Outer Sheath

Scissors Insert



38651ML

ROBI® KELLY Dissecting and Grasping Forceps, CLERMONT-FERRAND model, rotating, dismantling, with connector pin for bipolar coagulation, double action jaws, long, especially suitable for dissection, size 5 mm, length 36 cm including:

Plastic Handle, without ratchet

Outer Sheath

Forceps Insert



38651ON

ROBI® Grasping Forceps, CLERMONT-FERRAND model, rotating, dismantling, with connector pin for bipolar coagulation, double action jaws, with especially fine atraumatic serration, fenestrated jaws, size 5 mm, length 36 cm including:

Plastic Handle, without ratchet

Outer Sheath

Forceps Insert



38651MD

ROBI® KELLY Dissecting and Grasping Forceps, CLERMONT-FERRAND model, rotating, dismantling, with connector pin for bipolar coagulation, double action jaws, especially suitable for dissection, size 5 mm, length 36 cm including:

Plastic Handle, without ratchet

Outer Sheath

Forceps Insert

TAKE-APART® Bipolar Coagulation Forceps, 3 mm, deliver reliable coagulation in smaller patients.



26184HCS **TAKE-APART® Bipolar Coagulation Forceps**, with connector pin for bipolar coagulation, width of jaws 1 mm, size 3 mm, length 20 cm including:
Ring Handle
Outer Sheath
Forceps Insert

Connecting cable required for use with ROBI® and TAKE-APART® Bipolar Instruments



26176LE **Bipolar High Frequency Cord**, length 300 cm, for use with AUTOCON® III 300/400, AUTOCON® II 400 SCB system (111, 113, 115, 122, 125), AUTOCON® II 200, AUTOCON® II 80, Coagulator 26021B/C/D, 860021B/C/D, 27810B/C/D, 28810B/C/D, AUTOCON® series (50, 200, 350), Erbe-Coagulator, T and ICC series

A wide variety of ROBI® instruments is available.
See catalog: *Laparoscopy in Surgery, Gynecology and Urology*

Unipolar Instruments for Laparoscopic Interventions

The CLICKLINE series offers a range of unipolar instruments for laparoscopic and thoracic procedures. Available in a variety of jaw styles, with sizes 2 to 10 mm and lengths from 20 cm to 43 cm, CLICKLINE instruments are suitable for use in small, large and exotic animals.

CLICKline Instruments, unipolar, 3 mm



30321DBS

CLICKLINE BLAKESLEY Dissecting and Biopsy Forceps, rotating, dismantling, insulated, with connector pin for unipolar coagulation, with LUER-Lock irrigation connector for cleaning, single action jaws, size 3 mm, length 20 cm including:

Plastic Handle, without ratchet

Outer Sheath

Forceps Insert



30322MDS

CLICKLINE KELLY Dissecting and Grasping Forceps, rotating, dismantling, insulated, with connector pin for unipolar coagulation, with LUER-Lock irrigation connector for cleaning, double action jaws, size 3 mm, length 20 cm including:

Plastic Handle, with ratchet

Outer Sheath

Forceps Insert



30321MWS

CLICKLINE Scissors, rotating, dismantling, insulated, with connector pin for unipolar coagulation, with LUER-Lock irrigation connector for cleaning, double action jaws, serrated, curved, conical, size 3 mm, length 20 cm including:

Plastic Handle, without ratchet

Outer Sheath

Scissors Insert

CLICKline Instruments, unipolar, 5 mm



60421DZ

CLICKLINE Biopsy Punch, rotating, dismantling, insulated, with connector pin for unipolar coagulation, single action jaws, with two teeth, size 5 mm, length 43 cm
including:
Plastic Handle, without ratchet
Outer Sheath
Punch Insert



66321DH

CLICKLINE Biopsy Punch, rotating, dismantling, insulated, with connector pin for unipolar coagulation, single action jaws, through-cutting, size 5 mm, length 36 cm
including:
Plastic Handle, with ratchet
Outer Sheath
Punch Insert



66323BA

CLICKLINE BABCOCK Grasping Forceps, rotating, with connector pin for unipolar coagulation, single action jaws, atraumatic, with multiple teeth, fenestrated, size 5 mm, length 36 cm
including:
Plastic Handle, with hemostat style ratchet
Outer Sheath
Forceps Insert



66322ML

CLICKLINE Dissecting and Grasping Forceps, rotating, dismantling, with connector pin for unipolar coagulation, double action jaws, long, size 5 mm, length 36 cm
including:
Plastic Handle, with ratchet
Outer Sheath
Forceps Insert

CLICKline Instruments, unipolar, 5 mm



33326R

CLICKLINE Dissecting and Grasping Forceps, rotating, dismantling, insulated, with connector pin for unipolar coagulation, double action jaws, right-angled, size 5 mm, length 36 cm including:

Plastic Handle, with disengageable ratchet

Outer Sheath

Forceps Insert



66321MW

CLICKLINE Scissors, rotating, with connector pin for unipolar coagulation, double action jaws, serrated, curved, conical, size 5 mm, length 36 cm including:

Plastic Handle, without ratchet

Outer Sheath

Scissors Insert



66321DB

CLICKLINE Dissecting and Biopsy Forceps, rotating, with connector pin for unipolar coagulation, single action jaws, size 5 mm, length 36 cm including:

Plastic Handle, without ratchet

Outer Sheath

Forceps Insert

CLICKline Instruments, unipolar, 10 mm



33533DU

CLICKLINE DUVAL Grasping Forceps, rotating, dismantling, without connector pin for unipolar coagulation, double action jaws, size 10 mm, length 36 cm including:

Metal Handle, with hemostat style ratchet

Outer Sheath

Forceps Insert

A variety of CLICKline instruments is available.

See catalog: *Laparoscopy in Surgery, Gynecology and Urology*

Coagulation and Dissection Electrodes, unipolar, 5 mm



37370SC

Coagulation and Dissection Electrode, with suction channel, with connector pin for unipolar coagulation, size 5 mm, length 36 cm



37370DL

Coagulation and Dissection Electrode, with suction channel, insulated sheath, with connector pin for unipolar coagulation, L-shaped, size 5 mm, length 36 cm

Connecting cable for use with Unipolar Instruments



26005M

Unipolar High Frequency Cord, with 5 mm plug, length 300 cm, for AUTOCON® II 400 SCB system (111, 115, 122, 125), AUTOCON® III 300, AUTOCON® III 400, AUTOCON® II 200, AUTOCON® II 80, AUTOCON® system (50, 200, 350), Erbe type ICC and Erbe VIO 300

Neutral Electrode, size large, reusable



27805

Neutral Electrode, made of conductive silicone, with 2 rubber ties for fastening, contact surface A = 500 cm², for use with KARL STORZ AUTOCON® series (200, 350), AUTOCON® II 400 series (111, 115) with Connecting Cable 27806 as well as AUTOCON® III 400 and AUTOCON® III 300 with Connecting Cable 27806UR



27806UR

Neutral Electrode Connecting Cable, for use with Neutral Electrode 27805

Neutral Electrodes, size small, single use



27802

Neutral Electrode, contact surface divided into two, A = 169 cm², for single use, package of 50, for use with AUTOCON® 50/200/350, AUTOCON® II 400 all versions, AUTOCON® III 400 and AUTOCON® III 300, Connecting Cable 27806US required




27806US

Neutral Electrode Connecting Cable, for use with Neutral Electrode 27802


Electrodes for Arthroscopic Surgery

New Arthro VAP CUT Mode and Arthro VAP COAG Modes in the AUTOCON® III VET facilitate fast tissue ablation or shrinkage, with minimal heat emission to the surrounding tissues. Vaporization and hook electrodes are available in a variety of styles, sizes and angles.


Shaft Diameter 3.3 mm

- 


010153-10*


Hook Electrode, delicate, curved 90°, length 150 mm, STERILE 

Shaft Diameter 4 mm


- 

010152-10*


Vaporization Electrode, without suction, curved 70°, length 150 mm, sterile, for single use, package of 10, for use with Handle 28802HG/28802HS STERILE 

- 


010154-10*


Hook Electrode, curved 90°, length 150 mm, sterile, for single use, package of 10, for use with Handle 28802HG/28802HS STERILE 

Shaft Diameter 5 mm


- 


010151-10*

Vaporization Electrode, with suction, curved 90°, length 150 mm, sterile, for single use, package of 10, for use with Handle 28802HG/28802HS STERILE 


- 

010156-10*


Vaporization Electrode, without suction, curved 45°, length 150 mm, sterile, for single use, package of 10, for use with Handle 28802HG/28802HS STERILE 

- 

010157-10*

Vaporization Electrode, without suction, curved 90°, length 150 mm, sterile, for single use, package of 10, for use with Handle 28802HG/28802HS STERILE 

Handle required for use with vaporization electrodes

- 







28802HG

Handle for Vaporization Electrodes, reusable, for use with AUTOCON® III 300 and Vaporization Electrodes 010151-10, 010152-10, 010153-10, 010154-10, 010155-10, 010156-10, 010157-10, 010158-10



Flexible Electrodes

A variety of electrodes and snares to control hemorrhage and remove intraluminal masses is available for use with flexible endoscopes and rigid telescopes.

Electrodes, unipolar








	27770AA	Button Electrode , unipolar, 3 Fr., length 53 cm
	27770B	Button Electrode , unipolar, 5 Fr., length 53 cm
	67772A	Needle Electrode , unipolar, 5 Fr., length 53 cm
	60765L	Coagulation Electrode , unipolar, diameter 1.8 mm, working length 200 cm, for use with working channels as of size 2.2 mm
	61770AA	Coagulation Electrode , unipolar, 3 Fr., length 120 cm
	60330L	Coagulation Electrode , unipolar, diameter 1.8 mm, working length 350 cm, for use with working channels as of size 2.2 mm

Handle and connecting cable for use with flexible electrodes



	26520043	Electrode Handle , with 2 buttons for activating the unipolar generator, yellow button: unipolar cutting, blue button: unipolar coagulation, Connecting Cable 26520045 required
	26520045	High Frequency Cable , for Electrode Handle 26520043, length 400 cm, for use with AUTOCON® II 400 SCB 20535220-111 and 20535220-115

Flexible Snares

Snares with handles, unipolar

	67159L	Polypectomy Snare , unipolar, insulated, 5 Fr., length 32 cm	
	110220-10*	Oval Polypectomy Snare , diameter 30 mm, working length 180 cm, KARL STORZ HF connector, sterile, for single use, package of 10, for use with flexible endoscopes with working channels as of diameter 2 mm	
	110221-10*	Oval Polypectomy Snare , diameter 30 mm, working length 230 cm, KARL STORZ HF connector, sterile, for single use, package of 10, for use with flexible endoscopes with working channels as of diameter 2.8 mm	
	110225-10*	Asymmetric Polypectomy Snare , diameter 25 mm, working length 230 cm, standard HF connector, sterile, for single use, package of 10, for use with flexible endoscopes with working channels as of diameter 2.8 mm	
	60330LF	Polypectomy Snare , reusable, complete, with Handle 60774H, snare size 60 mm, asymmetric, diameter 2.4 mm, working length 350 cm, for use with working channels as of 2.8 mm	

Connecting cable for use with snares and flexible electrodes

	26005M	Unipolar High Frequency Cord , with 4 mm plug, length 300 cm, for use with Martin HF units	
	26002GI	Unipolar High Frequency Cord , with 4 mm plug and 3 mm bush on instrument side, length 300 cm	

Unipolar flexible electrodes and snares require the use of a neutral electrode.
See options and ordering information on page 13.
For additional resectoscopes and accessories see catalogs: *GYNECOLOGY, UROLOGY & MTP*



Instruments for Open Surgical Procedures

A selection of instruments designed for use in non-endoscopic procedures makes the AUTOCON® III VET even more versatile.



20530008 **Surgery Electrodes Set**
including:
Container with Lid and Sterilizing Insert,
for 16 electrodes with diameter 4 mm



Wire Snare, 5 mm



Same, 10 mm



Ribbon Snare, 10 mm



KIRSCHNER Spatula Electrode, straight



MAGENAU Knife Electrode, angled



Knife Electrode, lancet-shaped



Ball Electrode, 2 mm



Same, 4 mm



Same, 6 mm



Needle Electrode




Flat Electrode, 8 x 10 mm




Same, 10 x 15 mm

Accessories for Surgery Electrodes Set

Handle and connecting cable for use with surgery electrode set

- 

26520043


Button Electrode, with 2 buttons for activating the unipolar generator, yellow button: unipolar cutting, blue button: unipolar coagulation, High Frequency Cable 26520045 required
- 

26520045


High Frequency Cable, for Electrode Handle 26520043, length 400 cm, for use with AUTOCON® II 400 SCB 20535220-111 and 20535220-115

Unipolar surgery electrode set requires the use of a neutral electrode. See options and ordering information on page 13.

Bipolar Coagulation Forceps

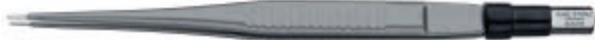- 


841216

Bipolar Coagulation Forceps, insulated, straight, blunt, tip 1 mm, length 16 cm, for use with Bipolar High Frequency Cords 847000 or 847000A/E/M/V
- 

842216

Same, angled tip

Connecting cable required for use with Bipolar Coagulation Forceps


- 

847000E

Bipolar High Frequency Cord, length 300 cm, for AUTOCON® II 400 SCB systems (111, 113, 115, 122, 125), AUTOCON® II 200, AUTOCON® II 80, Coagulator 26021B/C/D, 860021B/C/D, 27810B/C/D, 28810B/C/D, AUTOCON® systems (50, 200, 350), Erbe-Coagulator, T and ICC series and bipolar coagulation forceps

AUTOCON® III VET

System Components

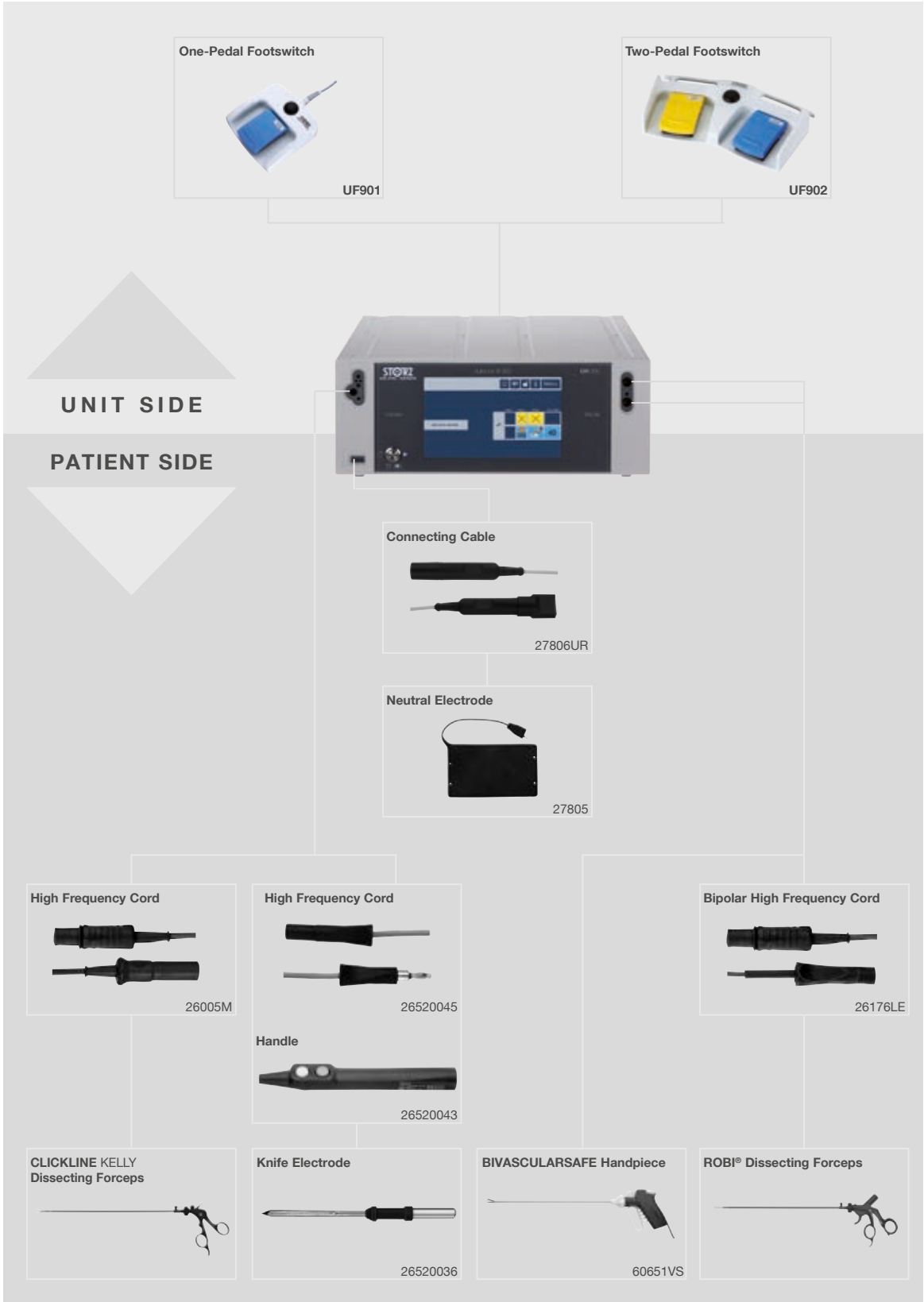

75 Years

*Shaping the Future
of Endoscopy with you*

STORZ
KARL STORZ — ENDOSKOPE
THE DIAMOND STANDARD

KARL STORZ SE & Co. KG
Dr.-Karl-Storz-Straße 34, 78532 Tuttlingen/Germany
Postbox 230, 78503 Tuttlingen/Germany
Phone: +49 7461 708-0
Fax: +49 7461 708-105
E-Mail: vet@karlstorz.com
www.karlstorz.com

