

Urology



For more information, please contact your distributor representative or KRUUSE USA at 1-833-578-8731 or email kruuse.usa@kruuse.com



BUSTER Foley Plus Catheter

Placement is easy with our highly biocompatible, wireless, 2-way catheter with retention balloon.

Perfect for:

- Drainage purposes
- Urine sampling
- Bladder lavage

Thoughtful user-friendly design

- Easy placement with no wire needed
- Radiopaque
- Size color-coding

Designed for patient comfort

- Soft and flexible
- High biocompatibility
- Suitable for long-term placement


Improved patient safety

The retention balloon secures the catheter in situ and the rounded tip and atraumatic side holes help avoid tissue damage. The highly biocompatible silicone makes the catheter suitable for long-term placement up to 7 days.


Easy insertion - no wire

The catheter is made of white silicone with an added substance that increases the stiffness of the catheter. This means that the catheter rigidity is increased and therefore does not require a wire for insertion.



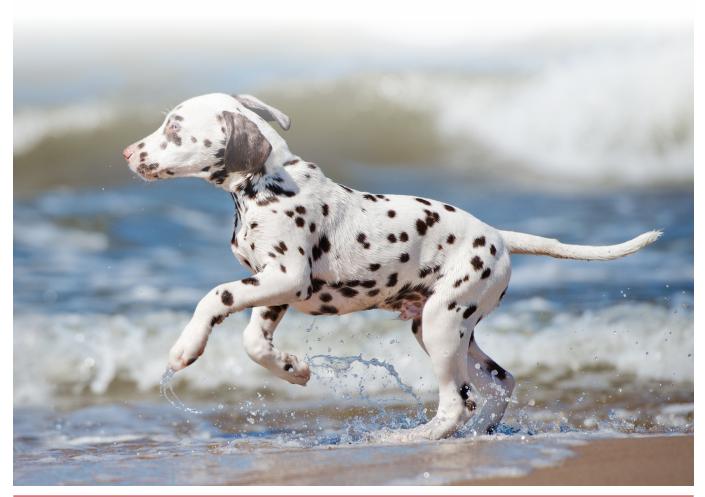



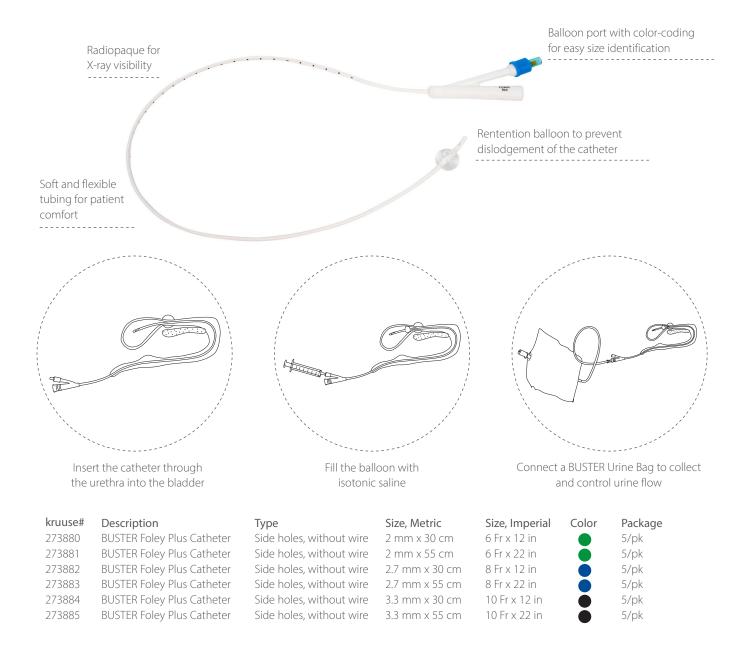





KRUUSE HC Urinary Catheter, sterile

(Hydrophilic Coated)

- Hydrophilic coating results in
 - A very smooth and viscous surface which reduces friction and prevents damage to the urethra
 - Uncomplicated catheterization; no need for using gel on the catheter
 - Coating is more evenly applied on the catheter compared to gel
- Atraumatic drainage holes significantly reduce the risk of a bladder infection
- Color-coded Luer lock fitting and label for easy size recognition
- User-friendly end-opening box and individually packed pouches
- DEHP-free (phthalate free) medical grade PVC

Directions for use:



Open the pouch



Pour suitable water in the pouch



Fix it on a tile if desired. Wait 30 sec



The catheter is ready for catheterization

Note: The catheter must not be left in the urethra for more than 10 minutes

A 1ST IN VETERINARY MEDICINE






BUSTER Easy Slide Cat Catheter

Easy and uncomplicated catheterization

Made from DEHP-free (phthalate free) medical grade PTFE (Polytetrafluorethylene), the BUSTER Easy Slide Cat catheter is available in two forms—with an open end or side holes. Both versions are suitable for drainage and/or bladder lavage.

- Smooth catheter for easier placement
- Perfect rigidity
- Softens at body temperature and takes shape of surrounding tissue
- Luer lock fitting allows for easy attachment of a syringe and urine bag
- Suturing holes in base plate for secure placement
- Radiopaque tubing for x-ray visibility
- Ideal for drainage of the bladder, hospitalized patients and patients undergoing surgery

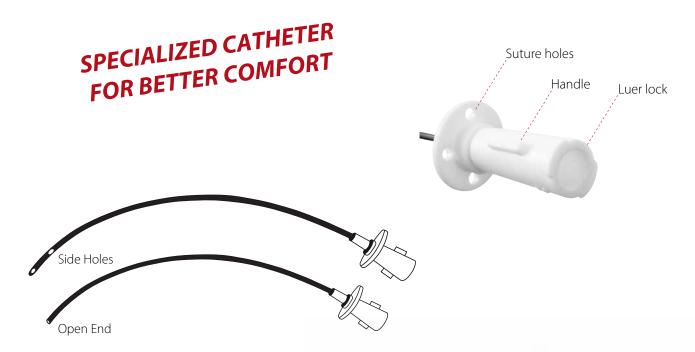

KRUUSE#	Туре	Size, Metric	Size, Imperial	Package
273400	Side holes	1.2 x 110 mm	3.5 Fr x 4 1/3 in	5/pk
273401	Side holes	1.2 x 140 mm	3.5 Fr x 51/2 in	5/pk
273436	Open end	1.2 x 110 mm	3.5 Fr x 4 1/3 in	5/pk
273437	Open end	1.2 x 140 mm	3.5 Fr x 51/2 in	5/pk
273438	Open end	1.2 x 180 mm	3.5 Fr x 7 in	5/pk





Dog Catheter Guide



Instant (<10 min)

- BUSTER HC Urinary Catheter



Short term (0-1 day)

- BUSTER Male Dog Catheter



Medium term (1-3 days)

- BUSTER NC Urinary Catheter



Long term (0-7 days)

- BUSTER Foley Catheter
- BUSTER Foley Plus Catheter



Instant (< 10 minutes)

Applications

- Urine sample collection
- Check of passage in urethra
- Bladder tension relief
- Bladder lavage

BUSTER HC Urinary Catheter

- ✓ No need for gel
- ✓ Soft and comfortable
- ✓ Color-coded hub and label for easy size recognition
- ✓ Luer lock on hub allows easy connection to syringe

Short term (0-1 day)

- Urine sample collection
- Check of passage in urethra
- Bladder tension relief
- Bladder lavage

BUSTER Male Dog Catheter

- More rigid catheter
- ✓ Luer lock on hub allows easy connection to syringe or urine bag

Recommended Catheters

Medium term (1-3 days)

Applications

- Urine sample collection
- Check of passage in urethra
- Bladder tension relief
- Bladder lavage
- Catheterization for drainage purposes during surgery or hospitalization

BUSTER NC Urinary Catheter

- ✓ Soft and comfortable
- ✓ Color-coded hub and label for easy size recognition
- ✓ Luer lock on hub allows easy connection to syringe or urine bag

Long term (0-7 days)

- Urine sample collection
- Bladder tension relief
- Bladder lavage
- Catheterization for drainage purposes during surgery or hospitalization

BUSTER Foley & Foley Plus Catheter

- ✓ Very soft silicone catheter with high biocompatability
- ✓ Color-coded hub and label for easy size recognition
- ✓ Urine drainage port fits KRUUSE urine bag
- Radiopaque for x-ray visibility

Catheter Guide



Flushing and short term (0-1 day)

- BUSTER Easy Slide Cat Catheter, open end
- BUSTER Cat Catheter, open end



Medium term (1-3 days)

- BUSTER Easy Slide Cat Catheter, side hole
- BUSTER Cat Catheter, side holes



Long term (0-7 days)

- BUSTER Barium Catheter



Flushing and short term (0-1 day)

Allows the vet or technician to flush uroliths retrograde into the bladder in the case of an obstruction

- Holes in the base plate allow suturing to the prepuce
- Luer lock on hub for easy connection to syringe or urine bag

open end

✓ Soft and flexible catheter

REMEMBER TO MONITOR URINE FLOW REGULARLY

Secommended Catheters

Applications

BUSTER Easy Slide Cat Catheter, BUSTER Cat Catheter, open end

- Optimal rigidity for easy catheterization
- ✓ Softens and adjusts to the anatomy
- Radiopaque for x-ray visibility

Medium term (1-3 days)

- Secure urine passage. Typically used in FLUTD/FIC cats, hospitalized patients and patients undergoing surgery
- The catheter has two lateral side holes and a rounded, atraumatically closed end
- Comes in various sizes with or without stylet
- Holes in the base plate allow suturing to the prepuce
- Luer lock on hub for easy connection to syringe or urine bag

BUSTER Easy Slide Cat Catheter, BUSTER Cat Catheter, side holes

✓ Optimal rigidity for easy catheterization

- ✓ Softens and adjusts to the anatomy
- ✓ Radiopaque for x-ray visibility

side holes

✓ Soft and flexible catheter

Long term (0-7 days)

Ideal for hospitalized patients and patients undergoing surgery

- The catheter has two lateral side holes and a rounded, atraumatically closed end
- Holes in the base plate allow suturing to the prepuce
- Luer lock on hub for easy connection to syringe or urine bag

Applications

Recommended

Catheters

BUSTER Barium Catheter

- ✓ Made of polyurethane and contains barium
- ✓ Radiopaque for x-ray visibility
- ✓ Polyurethane softens at body temperature increasing the biocompatibility. Hence, it is suitable for longer term placement and hospitalized patients

HOW TO REMOVE URETHRAL OBSTRUCTIONS IN CATS Scan the QR code

to read the article



RESOURCES

Webinar: Secrets to Successfully Manage Feline Urethral Obstruction, by Dr. Jody Lulich



Webinar: Current diagnostics and treatment options for cancer of the urinary tract in dogs, by Dr. Andy Roark and Dr. Sandra Bechtel



Scan the OR code to watch the webinar





Scan the QR code to watch the webinar

Scan the QR code to watch the video



BUSTER Easy Slide



Authorized US distributors



