



### **CompoSeal® Slim** Tube Sealing System

#### Flexible and efficient tube sealing technology

Versatile tube sealing platform for blood center, laboratory, and hospital use.

Cascade segment sealing with up to 8 devices

Quick and easy to set-up, use and re-configure

High-quality seals for reliable operations

Optional hand sealer adds flexibility and efficiencies

### CompoSeal Slim

Tube Sealing System

# Flexible desktop tube sealing system for single and cascade sealing


Combine up to 8 CompoSeal Slim devices for cascade sealing

- Adjustable segment length (60 120 mm)
- Visual alerts guide the operator through the sealing process. The sealing process is designed to seal from the last segment towards the blood bag to prevent pressure from increasing in the tube.
- One power cord needed per 2 devices

## CompoSeal Slim - versatile platform for high quality tube sealing

- Large range of sealable tubing (up to 5.9 mm outer diameter)\*
- Sealing time is approximately 1-2 seconds
- Integrated splash protection and tube positioning system
- LEDs display the status of the sealing process
- Hand sealer for additional flexibility and efficiencies (optional)
- Sealing head detaches for easy cleaning and disinfection



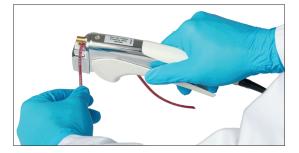

#### Optional hand sealer

-+

- The ergonomic design of the hand sealer supports ambidextrous use with low grip force to activate the sealing process
- Visual and acoustic alerts guide the operator through every seal

\*Does not apply to optional hand sealer

For more information, such as technical details, instructions for use, and training materials, please contact your local sales representative.

Please refer to the Operator Instructions for a complete list of warnings and precautions associated with the use of this device.



US Headquarters Fresenius Kabi Three Corporate Drive Lake Zurich, IL 60047 800-333-6925 Trademarks referred to are property of their respective owners. Copyright © 2020 Fresenius Kabi AG. All rights reserved.