


Clik-FIX™

Catheter Securement Devices

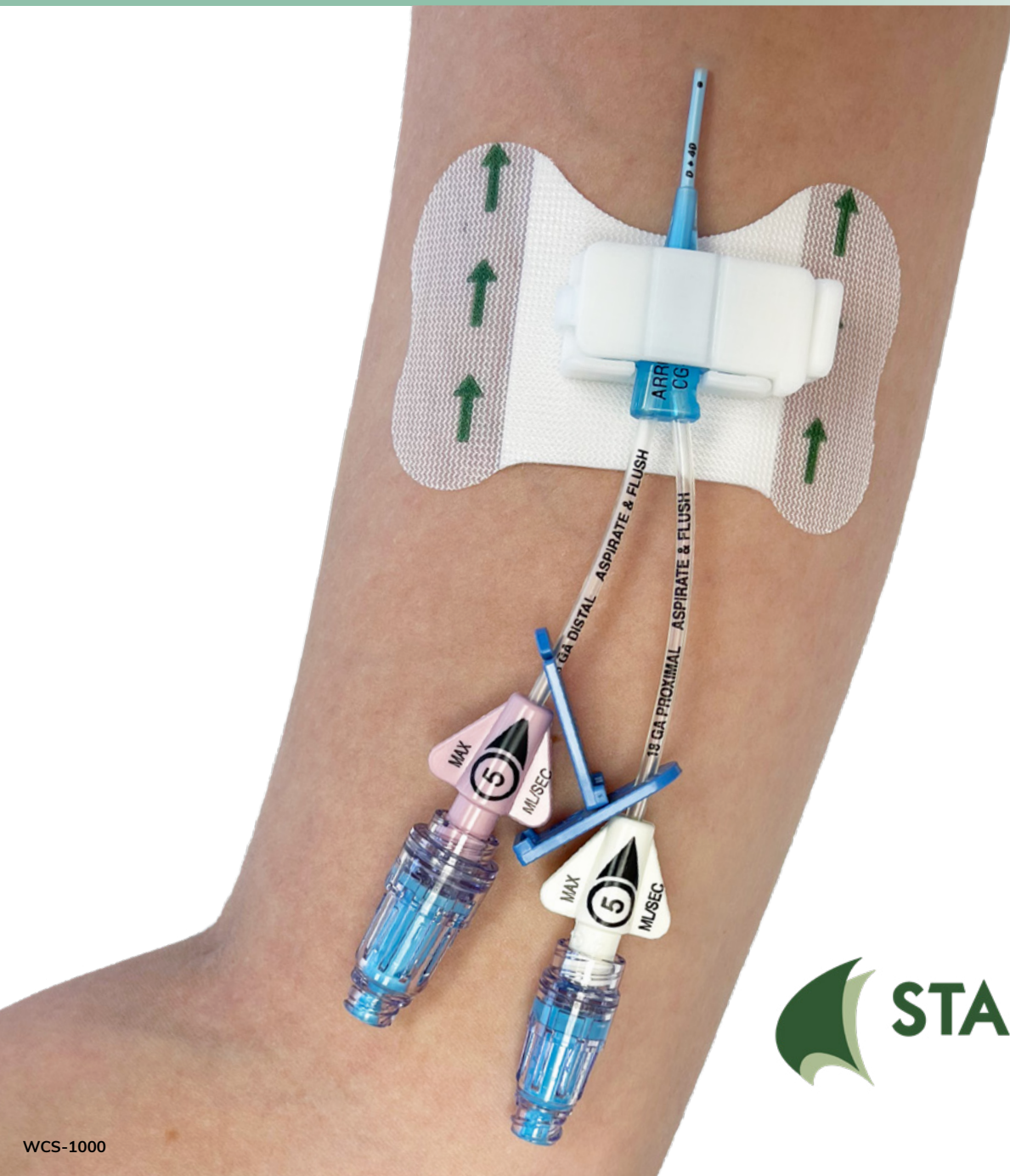


Starboard Medical's line of Klik-FIX™ catheter securement devices offer a variety of securement options that are all made with patient care in mind.

Better for the Patient

- Each device features a padded foam cushion for increased patient comfort, both protecting the skin and helping to lower the risk of pressure injuries
- Includes skin-friendly adhesive to help minimize skin irritation
- Devices are low-profile

Better for the Clinician

- Designed to be easy to apply and remove
- Engineered to secure better than market-leading devices
- Designed to address failure modes associated with other securement devices

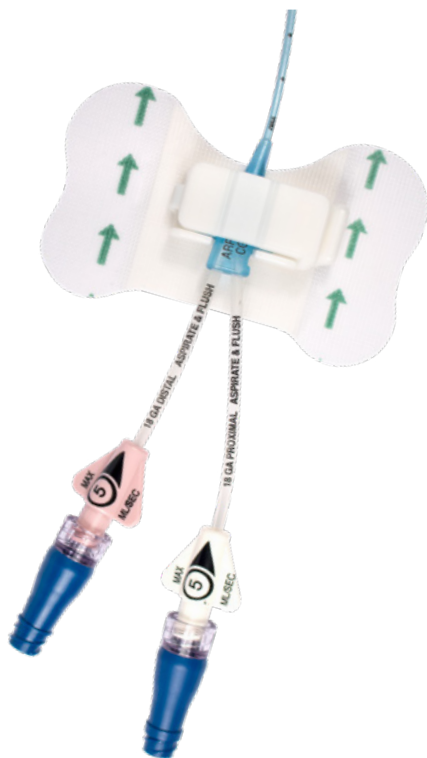

Klik-FIX
can withstand up to
13lbs.
of pull force¹.

Securing catheters with a securement device reduces complications and complies with standards and guidelines.


INS

“Use securement... in addition to the primary dressing, to stabilize and secure VADs. Inadequate securement can cause unintentional dislodgement and complications requiring premature removal.”²

CDC

“Use a sutureless securement device to reduce the risk of infection for intravascular catheters.”³


Securement devices can help prevent complications:

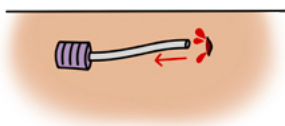


Dislodgement




CLABSI

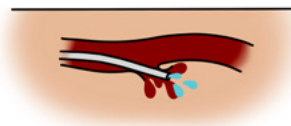


Infiltration

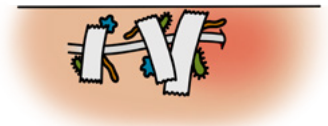


Infection

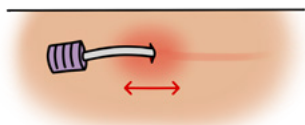

Phlebitis

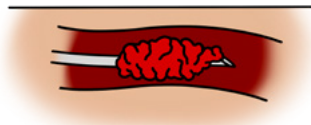

Occlusion

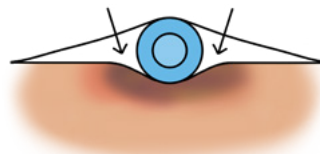


Pressure Injury



Malposition

Avoiding complications improves outcomes and costs.


Clik-FIX PICC/Central

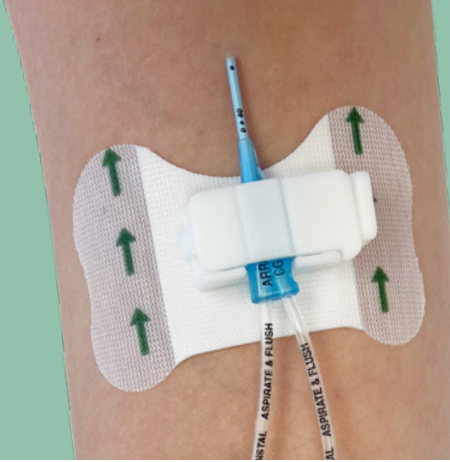


WCS-1000

Starboard Medical's Clik-FIX Catheter Securement Device makes securing a PICC or Central catheter easy. Designed to secure better, Clik-FIX PICC/Central securement features:

1. **Universal design:** device can secure the majority PICC, central, and midlines with wings. No need for multiple SKUs.
2. **Wing wall:** designed to help reduce the risk of dislodgement and micropistoning. Reduces micropistoning motion 650% better than the market leading device.
3. **Adjustable locking mechanism:** three adjustable positions to tightly secure a variety of catheters and provide stronger securement.
4. **Wing bed adhesive:** lightly holds catheter wing in place while securing. No need to line up suture holes to posts.
5. **Securement latch:** simple to use, simple to learn. Can open and close in one simple motion, yet is tamper-resistant.
6. **Skin-friendly adhesive:** more comfortable, more stable, and easier to remove.

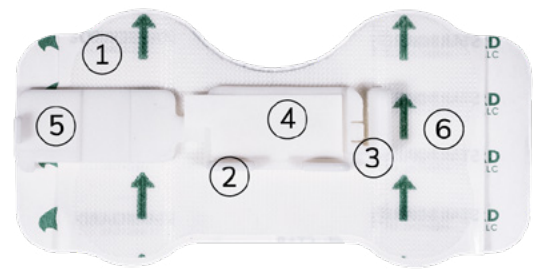


WCS-1000

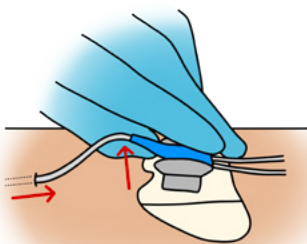

Competitor's Device

often requires upward movement to open. This can dislodge the catheter.

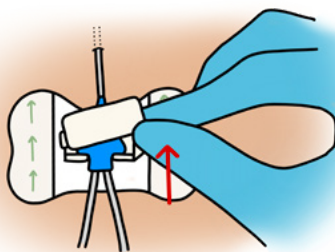

Clik-FIX Device

only requires a simple lateral nudge towards the insertion to open.

Unlike the competitor's hard to open gates, Clik-FIX features a secure yet easy to open design.

Catalog No.	Description
WCS-1000	Clik-FIX Catheter Securement for PICC and Central Catheters with Wings - Tricot Pad
WCS-1003	Clik-FIX Catheter Securement for PICC and Central Catheters with Wings - Foam Pad
50 Units Per Box	



WCS-1000
Tricot



WCS-1003
Foam



Scan to watch
educational video

Clik-FIX Soft Securement

WCS-1006



Starboard Medical's Clik-FIX Soft Securement device was specifically designed to offer more gentle, yet secure catheter securement. It is so gentle that we also offer it in a neonatal size. Designed to be comfortable and secure, Clik-FIX Soft Securement features:

1. **Soft foam base:** device has no hard plastic parts, only soft foam to cushion the catheter wings and help protect the skin.
2. **Chevron straps:** adjustable straps cross diagonally over the catheter wings to hold them tightly in place.
3. **Wing bed adhesive:** lightly holds catheter wings in place while securing. No need to line up suture holes to posts.
4. **Universal design:** straps are flexible and accommodating, and secure the majority of PICC, central, and midline catheters with wings.
5. **Hook-and-loop base:** allows securement straps to be repositioned and readjusted as needed.
6. **Skin-friendly adhesive:** more comfortable, more stable, and easier to remove.



WCS-1006



NPS-1006

The Clik-FIX Neonatal PICC device has a minimal footprint to be able to comfortably accommodate small neonatal patients.

Catalog No.	Description
WCS-1006	Clik-FIX Soft PICC/Central Securement
NPS-1006	Clik-FIX Neonatal PICC Securement
50 Units Per Box	



Scan to watch
educational video

Clik-FIX Peripheral

PCS-1011




Starboard Medical's Clik-FIX Peripheral Catheter Securement is all-in-one, replacing the need to use individual tape, gauze, cushion, and transparent dressing. Clik-FIX Peripheral securement features:


1. **All-in-one:** bordered bio-occlusive dressing and catheter securement. Less bulk, less fuss.
2. **U-shaped window:** provides a clear view of the insertion site and surrounding skin.
3. **Securement cover:** helps to reduce the risk of inadvertent catheter motion and dislodgement by nestling around the perimeter and back of the Luer lock. Easily re-positionable if access to the connection is required.
4. **Foam base pad:** cushions the load on the skin at the catheter hub/Luer lock junction and aids in maintenance of the catheter insertion angle.
5. **Solvent-free adhesive:** skin-friendly adhesive. Comfortable, stable, and easy to remove.
6. **Luer lock window:** allows for visualization of the catheter hub connection. Avoids downward pressure on top of the hub.
7. **Soft, low profile design:** no hard plastic parts to catch on things, bruise skin, or cause discomfort.



PCS-1011

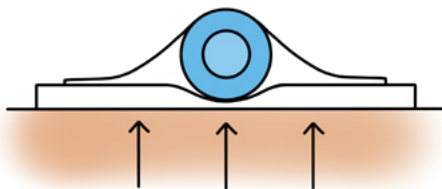

The foam cushion under the hub helps protect the skin from pressure injury

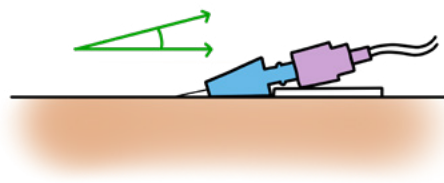


The foam cushion also aids in maintenance of the catheter insertion angle.

Catalog No.	Description
PCS-1011	Clik-FIX All 'n One Peripheral IV Securement
50 Units Per Box	



Scan to watch
educational video

Clik-FIX Universal

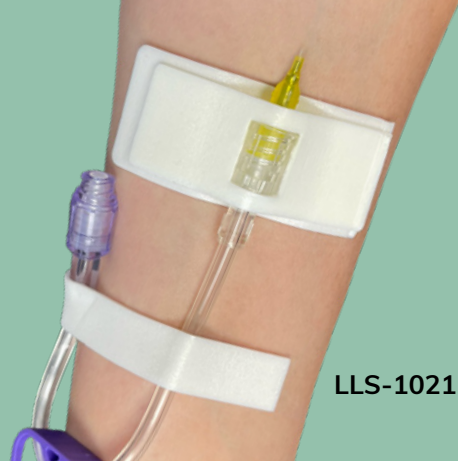


Starboard Medical's Clik-FIX Universal Catheter Securement secures around the catheter hub without adding additional pressure. Clik-FIX Universal securement features:

1. **Universal:** can secure any medical line with a Luer lock junction, as well as a variety of other catheters and medical tubing.
2. **Securement cover:** helps to reduce the risk of inadvertent catheter motion and dislodgement by nestling around the perimeter and back of the Luer lock. Easily re-positionable if access to the connection is required.
3. **Foam base pad:** cushions the load on the skin at the catheter hub/Luer lock junction and aids in maintenance of the catheter insertion angle.
4. **Solvent-free adhesive:** skin-friendly adhesive. Comfortable, stable, and easy to remove.
5. **Luer lock window:** allows for visualization of the catheter hub connection. Avoids downward pressure on top of the hub.
6. **Soft, Low profile design:** no hard plastic parts to catch on things, bruise skin, or cause discomfort.



LLS-1021

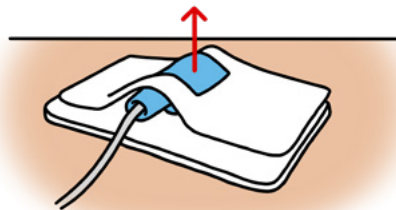

The window on the securement cover secures the hub without exerting downward pressure.

Catalog No.	Description
LLS-1021	Clik-FIX Universal Hub/Luer Lock Securement Device (Adult)
LLS-1024	Clik-FIX Universal Hub/Luer Lock Securement Device (Pediatric)
50 Units Per Box	



Scan to watch
educational video

Clik-FIX Foley

FCS-1073



Starboard Medical's Clik-FIX Foley Catheter Securement is made entirely out of cushioning foam and is soft, comfortable, and secure. Clik-FIX Foley securement features:

1. **Soft foam base:** cushions the catheter wings and helps to protect the skin.
2. **Flexible materials:** no hard plastic parts, only flexible materials to accommodate movement and catheter varieties.
3. **Hook-and-loop securement strap:** secures over the catheter and can be repositioned and readjusted if needed.
4. **Wye strap:** secures at the Wye junction to hold catheter tightly in place.
5. **Skin-friendly adhesive:** gentle on skin and easy to remove, yet secure.
6. **Center split release liner:** makes application easy.



FCS-1073

Catalog No.	Description
FCS-1073	Clik-FIX Foley Catheter Stabilization (16-24 French)
FCS-1074	Clik-FIX Foley Catheter Stabilization (10-14 French)
FCS-1075	Clik-FIX Foley Catheter Stabilization (5-8 French)
50 Units Per Box	



FCS-1073

FCS-1074

FCS-1075



Scan to watch
educational video

Clik-FIX Catheter Securement Devices are available directly through Starboard Medical or through your preferred distributor.

Please contact Starboard Medical for further product information, pricing, or samples.
Visit starboardmedical.com/catheter-securement for more information and resources.

Clik-FIX IV Securement

Catalog No.	Description
WCS-1000	Clik-FIX Securement for PICC/Central Catheters with Wings - Tricot Pad
WCS-1003	Clik-FIX Securement for PICC/Central Catheters with Wings - Foam Pad
WCS-1006	Clik-FIX Soft PICC/Central Securement
NPS-1006	Clik-FIX Neonatal PICC Securement
PCS-1011	Clik-FIX All 'n One Peripheral IV Securement
LLS-1021	Clik-FIX Universal Hub/Luer Lock Securement Device (Adult)
LLS-1024	Clik-FIX Universal Hub/Luer Lock Securement Device (Pediatric)
50 Units Per Box	

Clik-FIX Urological Securement

Catalog No.	Description
FCS-1073	Clik-FIX Foley Catheter Stabilization (16-24 French)
FCS-1074	Clik-FIX Foley Catheter Stabilization (10-14 French)
FCS-1075	Clik-FIX Foley Catheter Stabilization (5-8 French)
50 Units Per Box	

References:

Available upon request.

